# Gutta Cut®

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# Gutta Cut®

## Hot removal of gutta-percha points

### **Congratulations on purchasing your Gutta Cut®!**

This cordless device with interchangeable inserts is intended for fast, precise and safe cutting of excess gutta-percha points. The loop on the cutting tip provides easy access to the cavity and catches the excess gutta-percha exactly at the canal orifice and precisely cuts it off so that an unintentional loosening of the filling material is avoided. Various instrument tips are available and autoclavable for repeated use. The cordless handpiece can be used for up to about 80 applications without re-inserting it into the charging station and thus is always readily available for use.

### **Scope of supply**

- 1 Charging station  
(110 or 220 V versions available)  
with 2 sockets for 1 handpiece each
- 1 Handpiece (with re-chargable nickel-metal hydride battery)
- 4 Cutting tips
- 1 User instruction

Spare tips as well as a separate handpiece are also available from your supplier.

### **Technical specification**

Dimensions of handpiece:  
Length: 14.2 cm (without cutting tip), Ø: 1.8 cm  
Weight of handpiece: 42 g (without cutting tips)  
Mains voltage: 230V/50Hz; 110V/60Hz

### **Indication for use**

This instrument is intended for hot removal of gutta-percha points.

### **Counter indications**

No counter indications are known.

## Warning instructions

- In order to avoid risk of injury, do not let the tip come into contact with the lips, gums or mucous of the mouth of the patient. Always use rubber dam!
- Risk of fire: do not allow the heated tip or the handpiece to come into contact with combustible gases or liquids.
- Do not clean the instrument with a flammable cleaning solution.
- The instrument is not suitable for use in an environment containing flammable gases.
- Do not immerse the handpiece and the charging station in liquid.
- Please note that the cutting tips to be changed during a treatment may still be hot.
- If any liquid leaks out of the unit (possibly from the battery), stop work immediately and send the device to an authorised service centre for a change of battery. It is dangerous to use the device when the battery is leaking.
- It is dangerous to open the device for a change of battery. If you open the device, the guarantee will be voided.

## Precautions

- Never bend the tip while it is still in the handpiece.
- Using excessive force during treatment will have an adverse effect on the service life of the cutting tips.
- To safeguard optimal use of the nickel-metal hydride battery, never charge longer than 12 hours on end.

## Adverse reactions

No adverse reactions are known.

## Instructions for use

### Preparations

1. Please check before use that the mains voltage is the same as the voltage specified for the instrument.

### Charging

1. First, charge the instrument by connecting the charging station to a mains power socket.
2. Plug the handpiece into one of the two sockets in the charging station. While the instrument is charging, the diode lights up on the handpiece.

### Instructions for use with nickel-metal hydride battery

- Before using the instrument for the first time, make sure that it is fully charged. This process takes 12 hours.
- Always fully discharge the nickel-metal hydride battery in the Gutta Cut® before completely recharging it. This ensures an optimal life-span of your Gutta Cut®. Depending on the frequency of use, the product's life-span amounts to about 2 years.
- Avoid charging the battery longer than necessary for reaching full charging status. Never charge longer than 12 hours on end.

### Use

1. Fit a cutting tip onto the handpiece. Make sure that the angle of the tip is pointing in the opposite direction to the operating button. This makes work easier. If necessary, the 30° angle of the cutting tips can be slightly altered. However, this must be carried out with great care.

**Important:** Never bend the tip while it is still in the handpiece.

2. Insert the cutting tip into the cavity while it is cold (without pressing the operating button). The gutta-percha point to be cut off must be encircled by the loop of the cutting tip.
3. As soon as the cutting tip is located directly on the canal access, press the operating button.
4. The tip heats up in seconds and the gutta-percha point can be quickly and cleanly removed.
5. After the point has been cut off, release the operating button. The cutting tip immediately cools down and can be removed from the patient's mouth. The operating button is a „contact button“ which only switches the instrument on while it is pressed. When the button is released, the instrument switches off.
6. Remove the cutting tip from the handpiece after use.

## Cleaning and sterilisation

- Before each patient, the cutting tips must be cleaned and sterilised (even before using the instrument for the first time – the cutting tips are **not** supplied sterile).
- Any residues of gutta-percha which still remain on the instrument tip can be removed by carefully wiping it on a pellet of cotton wool.
- When used properly, the cutting tips can be autoclaved no more than 25 times. The sterilisation time depends on the sterilisation temperature (at 121°C, at least 20 minutes and at 132°C/134°C at least 5 minutes or 18 minutes for prion deactivation).
- Wipe the handpiece and the charging station with a mild, chlorine-free disinfectant. The disinfectant must be approved by the DGHM or FDA or bear the CE symbol. Apart from this, do not use disinfectant solutions which contain phenol, corrosion protection

agents or strong acid or strong alkaline solutions. When using disinfectant solutions, make sure that you follow the manufacturer's instructions.

## Maintenance

Gutta Cut® is maintenance free and simple to use.

## Troubleshooting

If the cutting tip does not heat up immediately after the operating button is pressed, check the electrical contacts on the cutting tip. If these are defective, change the cutting tip for a new one.

If the diode on the handpiece fails to light up during charging, check whether the charging station is connected to the mains or check the mains supply itself.

This medical product was put in circulation after 13 August 2005. The product may not be disposed of with normal household garbage. The disposal regulations for old electrical and electronic equipment must be followed.

**If you have further questions, please do not hesitate to contact us.**

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### Produsent: VDW GmbH
**Hersteller: VDW GmbH**