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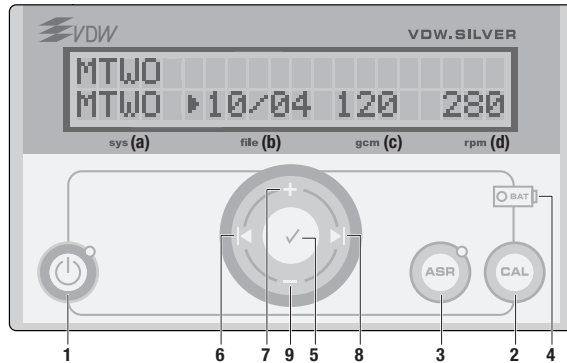
sys a) System (Mtwo, FlexMaster, Doctor's Choice)

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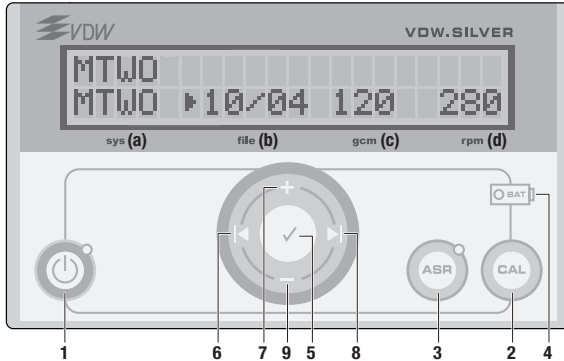
## Saving Dr's Choice settings

1. Press **◀** or **▶** to select the System Field
  2. Press **+** or **-** to select Dr's Choice
  3. Press **▶** to select the File Field
  4. Press **+** or **-** to select the desired file position
  5. Press **▶** to select the Torque Field
  6. Press **+** or **-** to select the desired torque
  7. Press **▶** to select the Speed Field
  8. Press **+** or **-** to select the desired speed
  9. Press **✓** to confirm/save the setting
- Pressing and holding the POWER and **✓** keys at the same time will reset factory default values.

## AUDIBLE TONES

1. When torque exceeds 75% of preset value
2. During reverse rotation of the micromotor

**To deactivate/activate audible signals:** Press **◀** and **▶** together

## Connections

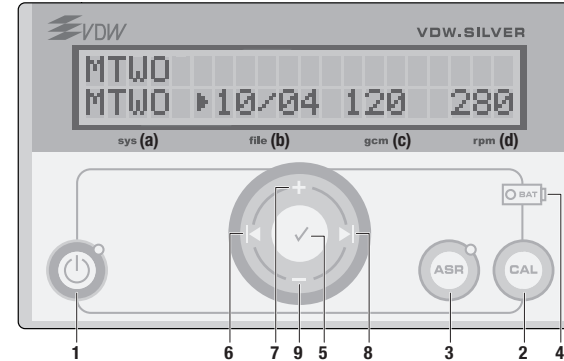
Battery Charger

Foot Control

Micromotor

## Keypad

1. **POWER:** on/off
2. **CAL:** Contra-angle calibration. Calibrate every time contra-angle is changed and at least once daily.
3. **ASR:** Green: auto-reverse on  
No LED: auto-reverse off  
Red: reverse
4. **BATTERY:** Green: battery OK  
Flashing red: battery needs to be charged  
Flashing orange: Charging
5. **✓ CONFIRMS:** modifications to torque/speed settings.  
Alternative to foot control: Starts rotation of file when pressed and held.  
Stop rotation by pressing any key.
6. / 8. **◀ / ▶:** Scrolls right/left in the display
7. / 9. **+ / -:** Scrolls up/down to select other systems or files or to change torque and speed in the Dr's Choice settings.

## Navigation in Display

Upper row: shows merely which NiTi system is in use (Mtwo, FM, Dr's Choice)

Lower row: using **◀** and **▶** it is possible to move right and left through the following:

sys (a) NiTi System (Mtwo, FlexMaster, Dr's Choice)

file (b) NiTi File (10/04, 15/05, 20/06, etc....)

gcm (c) Torque limit. (grams-force per centimetre. 1 g-cm = 0.0981 Nmm)

rpm (d) Rotation speed in rpm (revolutions per minute)

The active file is indicated with an arrow **▶**.

Using the **+** and **-** keys it is possible to scroll through the NiTi systems and files and to change the torque or speed values in the Dr's Choice file system.