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**Instructions for Use**

**Description:** DT Post removal kit is designed to remove DT Light Post® or DT White Post®. The kit contains 1 DT Pilot Drill and 1 DT Carbide Drill.

**Indications for use:** for root canal re-access where a DT Light or DT White Post® has been placed.

**Contra-indications:** none known to date - **Warnings:** for dental use only - **Precautions:** please disinfect and sterilise instruments before each use. Do not use hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) to disinfect the DT carbide drill. Use a rubber dam where possible.

**Adverse reactions:** none known to date

**Step-by-step instructions:**

1-5. Expose the post using a round bur.

2-6. Create a hole in the centre of the post with the pilot drill.

3-7. Mark the working length on the tungsten carbide drill with a silicone stopper and introduce the drill into the alignment hole. Drill at 800-2000 rpm until working length has been reached. Warning, this happens very quickly! Be careful to keep the drill centred (radiograph during the drilling process if possible) and not to exceed the working length.

4-8. If necessary, finish removal with the DT finishing drill #2 or #3.

**Other recommendations:** Do not use hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) to disinfect the DT carbide drill. The drill should not be used more than three times. Maximum speed of drill: 5000 RPM. The use of loupes (x5) is recommended.

**Safety and liability:** VDW declines any liability or compensation for possible damage due to the use of instruments which do not belong to the system or failure to observe the instructions for use. The user is responsible for ensuring that the products used are suitable.