EDDY™

de • en • fr • it • es

www.vdw-dental.com

VDW GmbH
Bayerwaldstr. 15 • 81737 Munich • Germany
Phone +49 89 62734-0 • Fax +49 89 62734-304
info@vdw-dental.com • www.vdw-dental.com

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en

EDDY Endo Irrigation Tip

With the EDDY Endo Irrigation Tip (hereinafter also referred to as EDDY) you have purchased a product that has been developed and tested with maximum care to meet the highest standards in terms of functionality and handling. These instructions for use are available in other languages upon request and on our website (<http://www.vdw-dental.com>).

1. Composition

The EDDY Endo Irrigation Tip is a sterile medical device made of polyamide.

2. Pack contents

5 blister packs, each containing 2 EDDY Endo Irrigation Tips, sterilised (REF V04 1441 000 000).

3. Symbols used

See table

4. Intended use

The EDDY Endo Irrigation Tip is designed to improve the cleaning of the root canal during the endodontic treatment by using a sonic-driven activator.

5. Contraindications

None known.

6. ⚠ Warnings

Read these warnings carefully before use. This section contains a description of potential side effects and safety risks for the product or user/patient.

- EDDY may only be used by qualified persons in dental practices or clinics.
- Always use a rubber dam, if not, the use of EDDY may constitute a health hazard to the patient.
- All persons in the treatment area must use eye protection.
- EDDY is used in conjunction with an air scaler handpiece that can be used for sonic-activated cleaning of the prepared root canal. The following air scalers are compatible with EDDY: W&H ZA-55, NSK TI-Max 970, KaVo SONICflex 2003. The use of air scaler handpieces that are not compatible with EDDY may lead to very serious consequences. You will find more information about compatible air scalers on our website.
- For warnings concerning the air scaler handpiece refer to the instructions for use of your air scaler.
- For the use of EDDY, set the frequency of your air scaler to the maximum performance level, whereby the upper limit of 6000 Hz must not be exceeded (air supply at a maximum of 0,3 MPa (3 bars)).
- EDDY may be bent slightly by the user but not kinked.
- Only activate EDDY when it is in the root canal.
- During irrigation of the root canal no additional manual pressure may be exerted on EDDY. If there is resistance, irrigation must be discontinued.
- In the case of an open apex, care must be taken to ensure that EDDY is inserted into the root canal only up to 1 mm short of the apex (observe the length mark).
- EDDY is a sterile single-use product. EDDY is only intended to be used on one patient. EDDY is not autoclavable nor disinfectable. If EDDY is reused, cross-contamination may occur.

7. ▽ Precautions

Read these safety instructions carefully before use. These precautions enable you to use the product safely and thus prevent injury to yourself and other individuals.

- Only use irrigation solutions commonly used in endodontics to irrigate the root canal, e.g. NaOCl solution (5.25% max.).

- Protect the patient's clothing.
- EDDY must be used properly in order to avoid injury and risks to the patient (see also section "Use of the EDDY Endo Irrigation Tip").
- For precautions concerning the air scaler handpiece, refer to the instructions for use of your air scaler.
- Safety and effectiveness in pregnant women and children have not been established.

8. Adverse Reactions

None known.

9. Use of the EDDY Endo Irrigation Tip

Step-by-step instructions

1. Use a rubber dam and eye protection.
2. Screw EDDY clockwise into the thread of the air scaler handpiece manually until the resistance to turning increases slightly. In doing so, make sure you do not bend or kink EDDY.
3. Introduce EDDY to the prepared root canal.
4. Fill the pulp chamber with a suitable irrigation solution for endodontics, e.g. NaOCl solution, EDTA.
5. Activate the air scaler as required for up to 30 seconds. In doing so, move EDDY carefully up and down with vertical motions. Do not exert any additional manual pressure. If there is resistance, discontinue irrigation.
6. Repeat irrigation with EDDY after every preparation step.

After completion of use: Remove EDDY from the air scaler handpiece by manually turning it counterclockwise.

10. Frequency of use (single use)

EDDY is a sterile single-use product (gamma-sterilised) and must be disposed of after treatment of one patient. If the tip should become kinked or deformed or if it shows signs of damage during treatment, discontinue the use of EDDY and insert a new EDDY into the air scaler.

11. Storage and disposal

The sterile EDDY Endo Irrigation Tip must be stored in the unopened blister pack. Protect from sunlight. After use, the EDDY Endo Irrigation Tip must be disposed of in accordance with the relevant national regulations and specifications.

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