m
two

Revisionen
Retreatments
Retratamientos
Reprise de traitement

VDW[®]

Endo
Easy
Efficient[®]

DEUTSCH 4

ENGLISH 6

ESPAÑOL 8

FRANÇAIS 10

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M_{two} Revisionsfeilen 15/.05 und 25/.05

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Feile: 15/.05: Max. Drehmoment 30 gcm (0,3 Ncm).

Feile: 25/.05: Max. Drehmoment 120 gcm (1,2 Ncm).

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Max. Torque 25/05 120 gcm (1,2 Ncm)

M_{two} direct: R15/05 Setting A. R25/05 Setting C.

- Open the canal orifice with a gates glidden drill or an ultrasound instrument and remove guttapercha from the coronal third. If necessary, soften guttapercha with a solvent (e.g. eucalyptus oil).
- Place the tip of the retreatment instrument on the guttapercha and allow it to rotate. M_{two} retreatment files have a cutting tip so that the instrument should progress easily in the guttapercha without pressure.
- Use M_{two} Retreatment instruments without downward pressure.
- Gradually remove the root canal filling with circumferential filing movements.

- Use a hand instrument (e.g. C-Pilot file) size ISO 08 to negotiate the root canal to full working length.
- Once full working length to the apex has been negotiated with a small, flexible hand instrument, prepare the root canal to size ISO 15 to full working length.
- Continue the preparation with conventional M_{two} instruments until the required size has been reached.

Preparation example:

M_{two} R15/05 or M_{two} R25/05, hand instruments ISO 08, ISO 10, ISO 15, M_{two} 10/04, M_{two} 15/05, M_{two} 20/06, M_{two} 25/06 etc.

For optimum safety, use a NiTi instrument only once. Always check instruments for signs of wear. As a general guide, use an M_{two} retreatment instrument in 3-4 canals if guttapercha is more difficult to remove and in 6-8 canals if guttapercha is easy to remove.

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VDW GmbH • P.O. Box 830954 • 81709 Munich, Germany