# SILVER POINTS

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In the USA: Rx only

#### 1.1 Product description

SILVER POINTS consist of 99.85% silver and 0.15% nickel. They are available in single sizes from ISO 10 to ISO 40 with a 2 % taper and 30 mm length and also as assorted size packs in ISO 15-40.

#### 1.2 Indications for use

SILVER POINTS are used as root canal filling material or as an optional x-ray accessory for length determination in endodontic treatments.

#### 1.3 Contraindications: None

## 2 General safety notes

#### 2.1 Warnings

SILVER POINTS contain 0.15 % nickel alloy. If used correctly, there is no known risk of allergic reaction.

#### 2.2 Precautions

The clinical application of SILVER POINTS for root canal fillings is limited to straight and round root canals.  
Choose compatible sealer in order to prevent corrosion of the silver point.

en

This product is intended to be used only as specifically outlined in these Directions for Use. Any use of this product inconsistent with these Directions for Use is at the discretion and sole responsibility of the dental practitioner.

#### 2.3 Storage conditions

SILVER POINTS should be stored at room temperature.

#### 2.4 Adverse reactions: None

## 3 Step-by-step instructions

#### 3.1 Cleaning and Disinfection

Before using SILVER POINTS, please clean with alcohol or an appropriate disinfectant solution, rinse and dry.

SILVER POINTS are for single-use only.

#### 3.2 Obturation

##### 3.2.1 Preparation of root canal

Prior to the application of a SILVER POINT, prepare, clean and dry the root canal to be filled, using the appropriate endodontic technique.

##### 3.2.2 Selection of appropriate SILVER POINT size

According to the prepared tip diameter of the last file used, the equivalent SILVER POINT is selected.

##### 3.2.3 Filling of root canal

The SILVER POINT is inserted with an appropriate sealer into the root canal until full working length is reached.

#### 3.3 X-ray accessory

Instead of a Hedstroem or K-file for taking an x-ray photograph, a SILVER POINT can be used for length determination by means of x-ray. The SILVER POINT is inserted into the root canal. An ISO size 10 is appropriate for sufficient radio-opacity.

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Manufacturer/Hersteller: VDW GmbH