



Endo
Easy
Efficient



EDDY
FLEX.CANNULA™
Irrigation Needle

Step by Step
Card



Preparation

Open the package by pulling along pre-notch provided for this purpose

Step by Step

Precautions:


Make sure that the needle can move freely to ensure constant flowback of the irrigation liquid.

This product has been manufactured under clean-room conditions. It is ready for use. Do not sterilize before use. Single-use device. Do not reuse.



Irrigation

1. Prefill your Luer lock syringe with one of the compatible fluids: stabilized sodium hypochlorite up to 6%, EDTA 17% or chlorhexidine digluconate 2%.
2. Insert the "Secure Lock" part of the needle into the male nose of your **previously filled** Luer lock syringe and lock the needle by turning it.
3. Make sure the needle is locked on the syringe.
4. Check the length of the canal to be irrigated and use the 18-, 19-, 20- and 22mm length markings to ensure proper positioning of the needle.

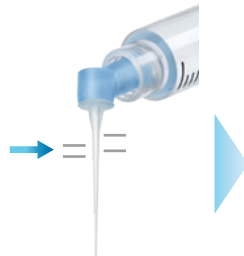



5. Gently insert the needle into the canal orifice and start irrigating at the coronal entry. Introduce the needle deeper into the canal until the coronal 2/3 of the canal can be irrigated abundantly. **Irrigate the canal with a continuous back and forth movement.**



Activation

Use EDDY® sonic powered activation tip to increase effectiveness of disinfecting solution.



Obturation

Dry canal and obturate.



Endo
Easy
Efficient

Distributed by:

VDW GmbH

Bayerwaldstr. 15
81737 Munich

Germany

Tel +49 (0) 89 62734-0

Fax +49 (0) 89 62734-304

info@vdw-dental.com



1639

Produits Dentaires SA

Rue des Bosquets 18,
1800 Vevey, Switzerland
pd-dental.com